

# IMMUVIEW®

## READER



### OBJECTIVE RESULTS


ImmuView® Reader is the qualitative reader for ImmuView® Family rapid tests that provides fast, objective results and allows clinicians to initiate the correct antibiotic treatment to improve the outcome for the patient.

ImmuView® is based on SSI Diagnostica's long tradition of premium quality antibody production.

**SSI Diagnostica A/S**  
Herredsvejen 2  
3400 Hillerød  
Denmark  
T +45 4829 9100

[ssidiagnostica.com](http://ssidiagnostica.com)


# INTRODUCTION TO THE IMMUIVIEW® READER

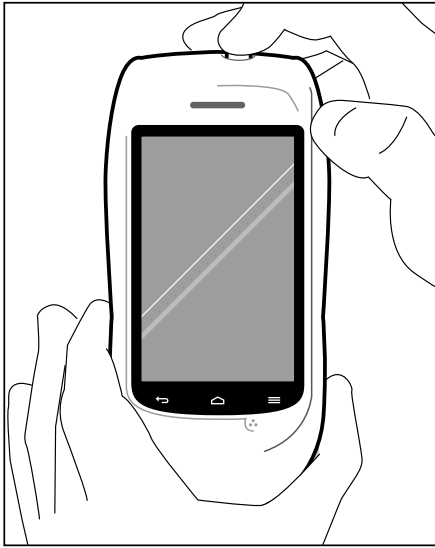


1. Turn on the ImmuView® Reader.



2. Press **Operator Data** and enter data. Press **OK**.

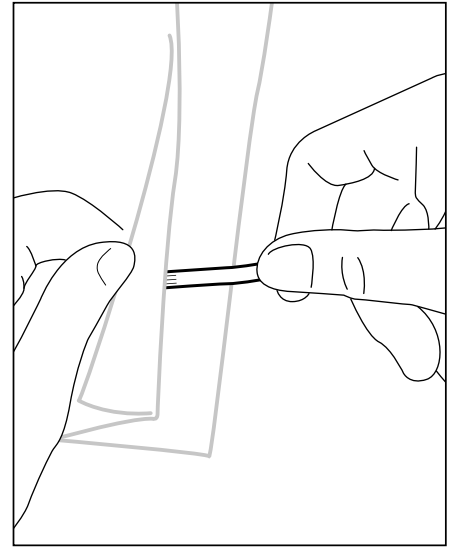

3. Dab off excess sample fluid on napkin using fingers or tweezers.

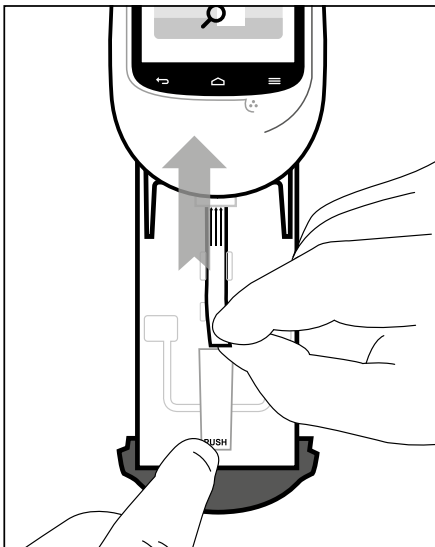

4. Insert test. Open drawer, insert test and close drawer.




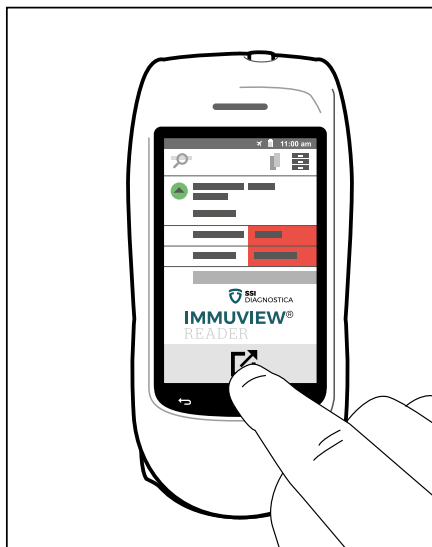
5. Press **Read Sample**.




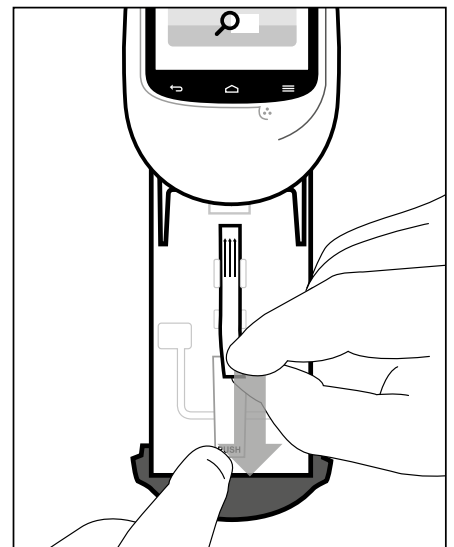
6. Read test results.



7. To save results press 



8. To export results press 



9. Discard test. Open drawer, remove test and close drawer.